


PRODUCT DRAWING

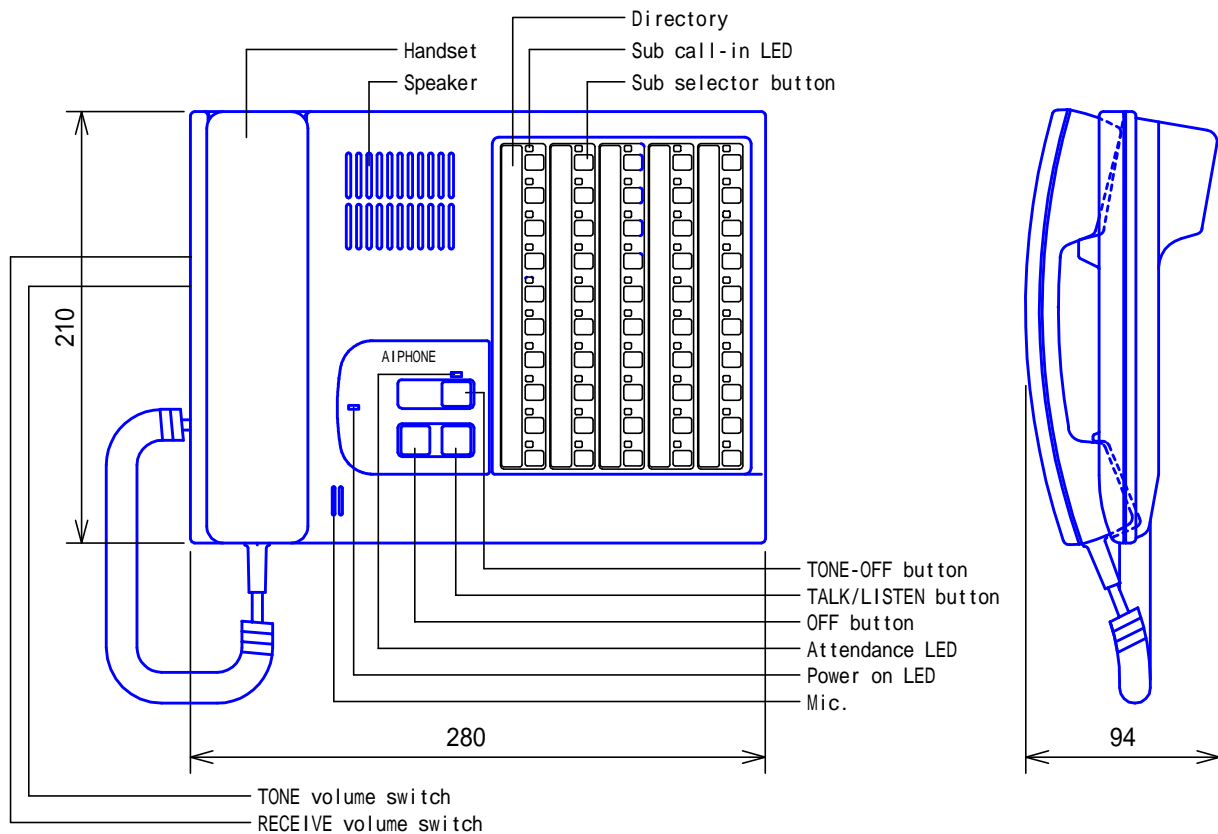

MOUNTING

NHX-50M is sdely used on desk-top.


SPECIFICATIONS

Power source	Supplied from CCU (NHX-80X).
Current consumption	500mA(max.). 80mA(in standby).
Communication	Open voice press-to-talk, release-to-listen, or Handset auto voice actuation.
Capacity	50 Sub stations.
Mounting	Desk-top use.
Housing	Fire-retardant plastic.
Unit color	White
Remarks	Certification; UL1069 Listed. C-UL Canadian Standard C22.2

DESCRIPTION MASTER CONSOLE STATION (50-call)	FIG. NAME PRODUCT/SPECIFICATIONS	UNIT mm	DATE 23 May, 2005
MODEL NO. NHX-50M	FIG.NO. 05414301-1-2	PAGE 1/2	REVISION 1

AIPHONE

WIRING DIAGRAM

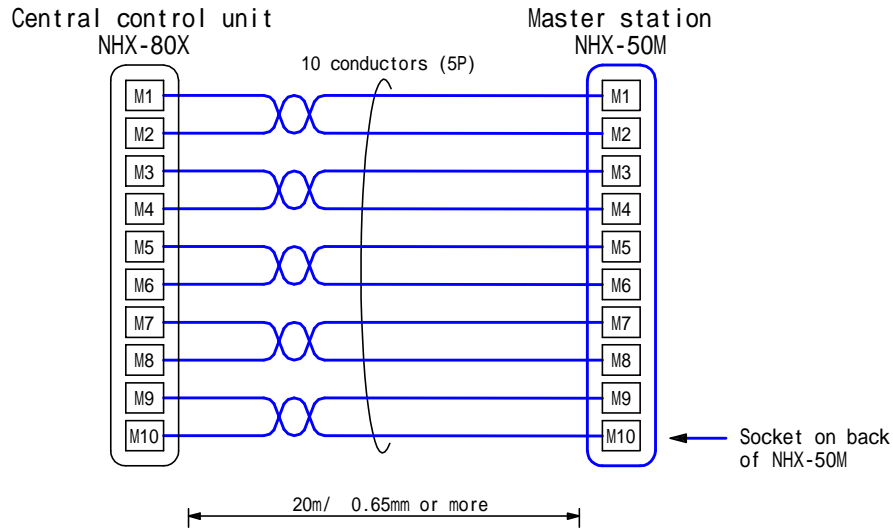

RESTRICTIONS

For wiring between NHX-50M and NHX-80X, use a supplied terminal cord of approx. 2m long. Plug in 10-pin socket at NHX-50M end. Following the Wiring diagram, wire M1~M10 terminals in five polarized pairs.

FUNCTIONS

Receive a call from Sub stations and enter communication. The communication is carried out either by lifting handset (auto voice actuation) or by using TALK button (open voice press-to-talk).

The call is classified into Routine call (bedside), Urgent Bathroom call, Cord-out trouble call, which can be identified by chime tone sounding and blinking LED.

Receiving Line-off trouble alarm.

(1. Bedside sub from Trunk. 2. Duty station from CCU)

SETTING

Adjust 5 preset volumes with controls on back of NHX-50M;

1. PRE-TONE. 2. TRANSMIT 3. TONE 4. DUTY STATION 5. RECEIVE.

Adjust RECEIVE & TONE volumes on the left side of unit;

Each 3 positions, HIGH - MID - LOW

DESCRIPTION MASTER CONSOLE STATION (50-call)	FIG. NAME WIRING/RESTR./FCN./SETTING	UNIT mm	DATE 23 May, 2005
MODEL NO. NHX-50M	FIG.NO. 05414301-2-2	PAGE 2/2	REVISION 1

AIPHONE